



HPV- home test

- Keep the test out of reach of children.
- Do not use the test after the expiration date is exceeded.
- Store at a temperature of 4-28 °C.
- All components are for single use.


Testing at home are your specialists in the field of home testing and allows for continuous development for new tests.

Look for the latest developments on
hometestshop.co.uk



Microbiome Ltd.
www.microbiome.eu

Brush
ROVERS MEDICAL DEVICES B.V.
Lekstraat 10
5347 KV Oss, NL

Sealbag 
Transposafe Systems Holland B.V.
Wattstraat 7a
2171 TP Sassenheim, NL

In-vitro for self use.



HPV- home test



HPV- home test

User manual


Read this manual
carefully before
starting the test.



User manual HPV-home test

This test detects the human papillomavirus. The human papillomavirus, also known as HPV, is the virus that exists on the skin and mucous membranes.

HPV can lead to cervical cancer. This home test shows whether the papillomavirus is present in the clinical samples you submit. If you test positive you should contact your general practitioner.

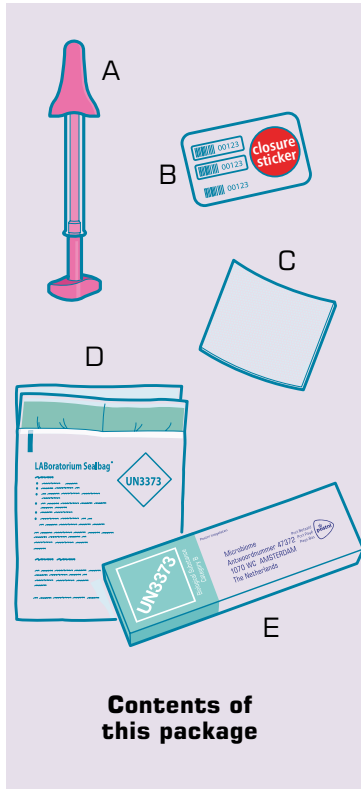


Contents of this package

- A. Brush
- B. Code card with stickers and your login code for the test results
- C. Absorption material (may remain in the plastic bag D)
- D. Plastic bag
- E. Pre-paid postage return box


For women: vaginal swab sample (wash your hands before usage)

Women should send a vaginal swab. For the vaginal swab you should use brush A. Remove the cap from brush A. Ensure that you do not touch the white fibers of the brush with your hands.

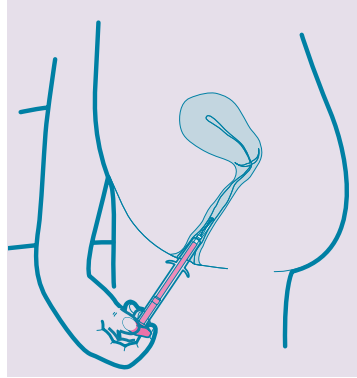


Position yourself comfortably, similar to when you are inserting a tampon. Spread your labia with one hand, and with the other, insert the brush into your vagina. Push the pink plunger in the direction of the transparent casing, until you hear a click. Turn the pink plunger five rotations in the same direction.

Carefully remove the brush from the vagina and pull the plunger so that the white fibers are in the casing. Press the cap firmly on the brush.

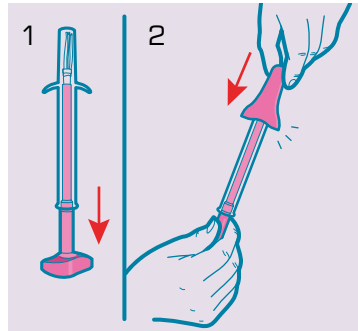

Important: coding the sample (clinical material)

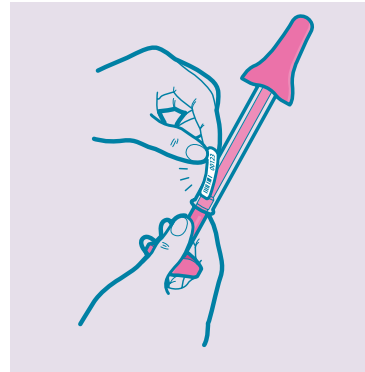

On the enclosed code card B is a code sticker that has to be placed on brush A. Your test result is linked to this unique code. **Do not forget this and properly store your code card with login code** so you can get your test results through the website.

Sending the sample

A) Place brush A and absorption material C in plastic bag D. Remove the silver foil strip from the adhesive edge and seal the plastic bag tightly. Place the filled plastic bag back into the pre-paid postage return box E. Close this box with the round seal located on code card B. The return box is freepost and has a return address on it, so you do NOT need stamps to send it.

Retrieving your results

The shipment usually takes 2 days and analyzing the sample will take up to 5 business days, including administrative processing. After 7 days, you can request your result via the website:

- Go to www.microbiome.eu
- Click 'Microbiome home tests' and then 'test results'
- You can then enter your login code in the text field and the test result will appear
- **Please click on OK** after you have seen your test results, so we know that you have seen it.